



Knee Preservation System™

GENESYS™ Matryx® Interference Screw




GENESYS™ MATRYX®
INTERFERENCE SCREW



GENESYS™ MATRYX®
SCREW TAP



GENESYS™ MATRYX®
TRI-LOBE DRIVER

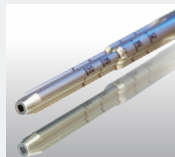

Learn more about
the GENESYS™
Matryx® and other
innovative products.
Call 800-237-0169
or visit linvatec.com.




GENESYS™ Matryx®
Interference Screw

microTCP.™ The GENESYS™ Material utilizes a patented micro filtration process yielding β -TCP particles to their smallest form, known as microTCP.

Strength. These proprietary microTCP particles are uniformly embedded throughout the 96L/4D PLA bioabsorbable polymer, delivering a more porous and stronger biocomposite material.¹

Ingrowth. GENESYS™ Matryx® provides reliable fixation during the critical healing period. It is followed by resorption exposing a scaffold which enables bone ingrowth to aid in the restoration of the patients' natural anatomy both biologically and mechanically.¹

Graft Fixation Strength

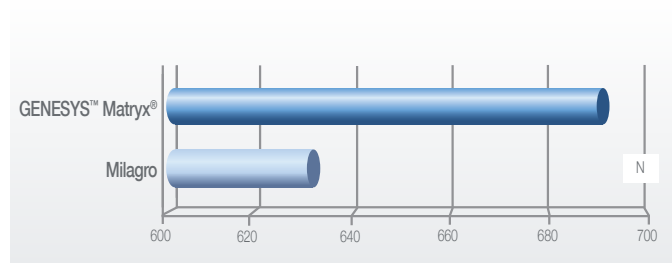

*Data on file.

As the chart demonstrates, **Graft Fixation Strength** performance of the GENESYS™ Matryx® Interference Screw far surpasses the Mitek Milagro.

¹Dalcusi, Guy, et al. Long Term Study of Bone In-Growth Process at the Expense of Poly(96L/4D-lactide)/beta-tricalcium Phosphate Composite. INSERM U791: Osteoarticular and Dental Tissue Engineering Research Center. 2011.

Low Profile Rounded Head


Reduces trauma to graft and improves femoral aperture fixation

Thread Tip Design

Threads extend to the distal tip, easing screw insertion

Broad Range of Sizes

Largest amount of size options indicated for ACL/PCL fixation

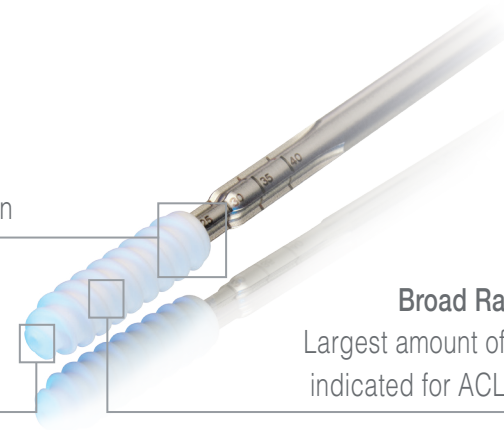

GENESYS™ Matryx® Interference Screw

width (mm)	5	5.5	6	6.5	7	8	9	10	11
15	235015M5	235515M5	236015M5	236515M5	–	–	–	–	–
20	235020M5	235520M5	236020M5	236520M5	237020M5	238020M5	239020M5	231020M5	231120M5
25	235025M5	235525M5	236025M5	236525M5	237025M5	238025M5	239025M5	231025M5	231125M5
30	235030M5	235530M5	236030M5	236530M5	237030M5	238030M5	239030M5	231030M5	231130M5
35	–	–	–	–	–	238035M5	239035M5	231035M5	231135M5
length (mm)									

TRI-LOBE DRIVER

5mm Fixed	D8652
5mm Short Modular	D8650
5mm Extended Length Modular	D8660
5.5–6.5mm Fixed	D8653
5.5–6.5mm Short Modular	D8651
5.5–6.5mm Extended Length Modular	D8661
7–11mm Fixed	DFS70
7–11mm Short Modular	DMS70
7–11mm Extended Length Modular	C8716

TAPS

7–8mm Fixed	TFS70
7–8mm Short Modular	TMS70
7–8mm Extended Length Modular	D8607
9–10mm Fixed	TFS90
9–10mm Short Modular	TMS90
9–10mm Extended Length Modular	D8609
11mm Fixed	TFS11
11mm Short Modular	TMS11
11mm Extended Length Modular	D8611

INSTRUMENTATION ACCESSORIES

Universal Driver, Modular Ratcheting Handle	D8640
Nitinol Guidewire 0.8mm dia. x 356mm length (0.032" x 14") for use with 5mm-6.5mm interference screws (5 per box)	C8006
BioScrew® HyperFlex® Nitinol Guidewire 1.1mm dia. x 356mm length (0.045" x 14") for use with 7mm-11mm biocomposite and bioabsorbable interference screws (5 per box)	C8026



ConMed Linatec
Knee Preservation System™

GENESYS™ Matryx® Interference Screw

11311 Concept Boulevard
Largo, Florida 33773-4908

Local 727-392-6464
Toll Free 800-237-0169

Website linatec.com
Email customer_service@linatec.com