



SutureLoop™ Whip Stitch Technique

Graft preparation using the SutureLoop Whip Stitching technique significantly reduces graft preparation time. The #2 Hi-Fi® SutureLoop is offered with options for straight or curved needles.

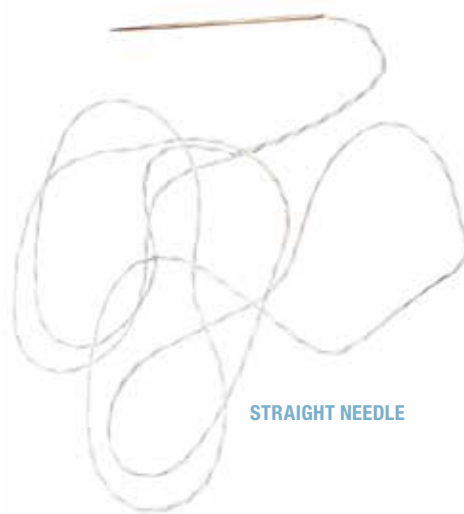


STRAIGHT NEEDLE

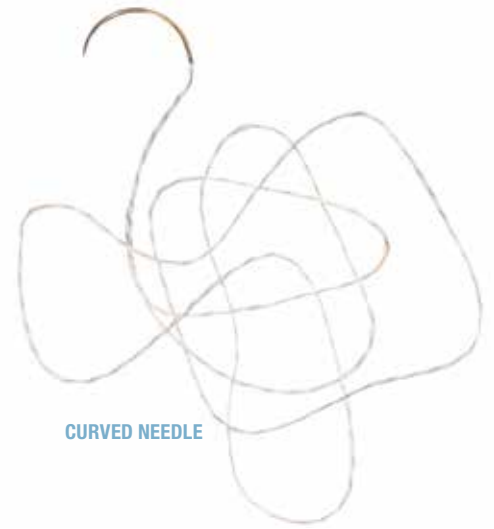


CURVED NEEDLE


1 Secure one end of the double or single soft tissue graft bundle to the graft prep table. Hold the opposite end of the graft by hand or using a clamp so that the graft end can be easily passed through the suture loop.

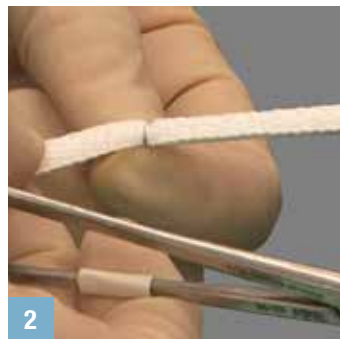

2 Pass the center of the SutureLoop over the free end of the graft and position it at the desired proximal starting point.

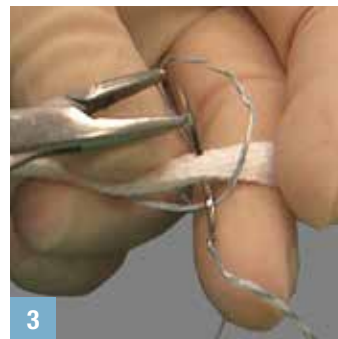


3 Pass the needle through the graft.

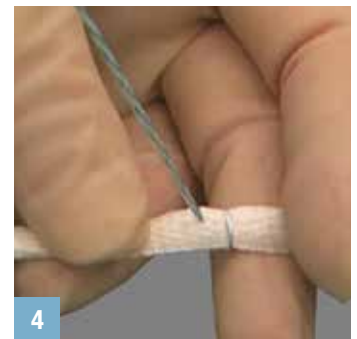


4 Tension the stitches.



5 Spread the SutureLoop strands and pass the graft tail through the loop.

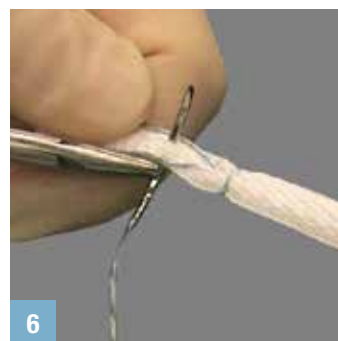

6 Pass the needle through the graft distal to the first stitches. Continue stitching toward the end of the graft.

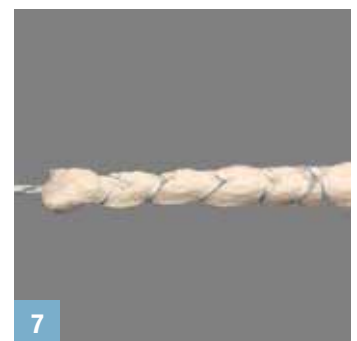

7 Continue the process stitching toward the end of the graft until the desired whip stitch length.

Cut the needle off leaving free ends or leaving tails braided together.

View video:

ConMed.com/mobile/videos/SutureLoop_WhipStitching_STV.html