

Sequent™ Meniscal Repair Device PRDT™ Technique




PIERCE

Pierce completely through the meniscus.




ROTATE

Rotate the device two full revolutions.

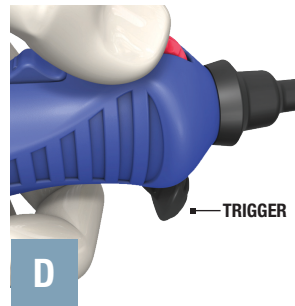

DEPLOY


1. Advance the trigger forward to ready the device.
2. Pull the trigger to deploy one implant.
3. Withdraw the needle from the meniscus.

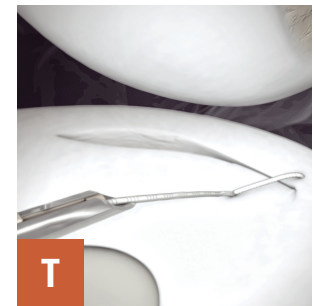


TENSION

To tension the stitch, pull back on the device and reel in suture with the red thumb wheel.

CHANGING PORTALS: BAILOUT TECHNIQUE

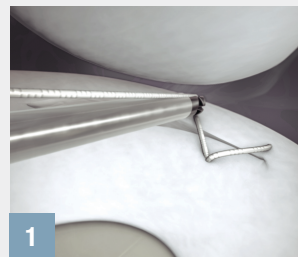

1 Cut the suture tail flush.

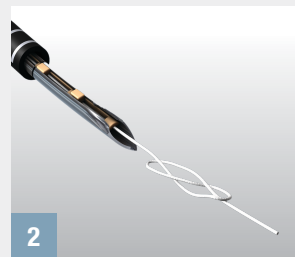

2 Tie a figure-of-eight knot.



3 Reel in the suture until the knot sets against the first implant. Do not over tighten.

NOTE: A minimum of two stitches must be used for each repair. Ensure enough implants are left when using the "Changing Portals" technique.



Sequent™ Anchors are comprised of PEEK-OPTIMA® polymer from Invivo® Biomaterial Solutions. PEEK-OPTIMA and INVIVO are registered trademarks of Invivo Ltd. All Rights are Reserved.



This product contains Dyneema® Purity. Dyneema® Purity is a registered trademark of Royal DSM N.V.